



**CHARACTERISTICS**

**IMOpro NATURPHRASIA**® from the **IMOpro**™ line, is a Medical Device for ophthalmic application consisting of a sterile isotonic saline solution, with physiological pH, non-viscous, preservative-free, with herbal distilled waters (including Bilberry), Hyaluronic acid and vegetable glycerine.

**IMOpro NATURPHRASIA**® eye drops are packed in a box containing 10 units single-vials 0,5 ml, protected by a foil pouch. All the distilled waters used in the **IMOpro NATURPHRASIA**® formulation are known for their refreshing properties and help to relieve from ocular redness and discomfort. In particular, the distilled water of blueberry contains anthocyanins and flavonoids that perform a protective action against the oxidative damages to which the ocular surface is exposed on a daily basis. The distilled waters used for the formulation are extracted in a current of steam and have no preservatives. The hyaluronic acid is a highly biocompatible polysaccharide polymer consisting of N-acetylglucosamine and glucuronic acid. In the eye, it can be found - for example - in the vitreous and aqueous humour. The hyaluronic acid has a great capacity for holding water molecules and, in an hydrated state, shows a visco-elastic behaviour which makes it an excellent moisturizer and lubricant.

The vegetable glycerine is a hygroscopic element (moisture-absorbing), highly biocompatible, with great humectant and wetting properties. At ocular level it acts as a lubricant since - causing the lowering of the vapor pressure of the tear liquid, - it allows the upper eyelid to easily slide on the cornea, thus minimizing friction during blinking.

**INDICATIONS**

**IMOpro NATURPHRASIA**® formula is intended to moisturize, lubricate, refresh and therefore for the soothe several situations of ocular discomfort, as specified below.

The combined action of the natural components contained in the formulation relieves from the redness, fatigue and discomfort of the eyes and ocular mucosa, caused by environmental factors (such as pollution, smog, prolonged exposure to sunlight, salt water, chlorine water, wind, dust, smoke) or by visual stress due to reduced blinking mainly caused by prolonged use of computer, intense study and long-time driving.

Thanks to its carefully selected components (and especially due to the combination of hyaluronic acid and glycerine), **IMOpro NATURPHRASIA**® formulation is suitable for those who suffer from mild and occasional ocular dryness and brings relief to the eyes, performing a moisturising and lubricating action.

The use of **IMOpro NATURPHRASIA**® before wearing contact lenses facilitates their application; moreover, the instillation after their application makes them more comfortable, contrasting the feeling of dryness.

**IMOpro NATURPHRASIA**® formulation is preservative-free and is also suitable for frequent and prolonged use.

**INSTRUCTIONS FOR USE:** **1.** Open the protective foil pouch. **2.** Separate the single-dose vial from the strip of 5 units. **3.** Twist and snap off the cap of the vial. **4.** Gently press the monodose vial and deliver 2/3 drops per eye. **5.** After application, immediately close the vial with its cap and use the content left over within 12 hours. **6.** Store the unused vials in their original foil pouch. **7.** The product is single-patient. **8.** The product can be used with contact lenses.

**COMPOSITION:** Bilberry distilled water, *Hamamelis virginiana* distilled water, *Chamomilla recutita* distilled water, *Euphrasia officinalis* distilled water, Sodium Hyaluronate, Vegetable Glycerine, Sodium chloride, Disodium hydrogen phosphate dodecahydrate, Sodium dihydrogen phosphate monohydrate, purified water q.b. up to 100 ml..

**PACKAGING:** The box is composed of 2 foil pouches each one containing a 5 vial monodose strip: each vial is 0,5 ml and re-closable.

**WARNINGS:** • The product is for ophthalmic use only • Do not use the product if the packaging is damaged or the bottle is not perfectly intact • Keep out of reach of children • Carefully clean hands before opening the vial • Do not touch the eye or other surfaces with the nozzle of the vial during installation • Do not use the product in case of known incompatibility to even one single component of the formulation • If necessary, check with your optics for the compatibility of the eye drops with the contact lenses used • In case of disturbances during use, immediately suspend the treatment and contact a doctor • Any administration of ophthalmic drug/s should be at least 15 minutes after the instillation of these eye drops, especially if the products contain benzalkonium chloride, or other quaternary ammonium salts, to avoid the precipitation of hyaluronic acid • The administration of ophthalmic drugs concomitantly with these eye drops should be carried out under the supervision of a doctor who can assess the possible interactions • Do not use the product after the expiration date indicated on the package • Do not disperse the container in the environment after use • Store in a dry place, away from heat sources and direct exposure to sunlight, at a temperature not above 30°C.

**VALIDITY:** 36 months with the package intact.

**MANUFACTURER:** Enable Innovations S.r.l. - Via Claudio Tolomeo n° 20/4 - I - 41012 Carpi (MO), Italia

**DISTRIBUTOR:** IMO SpA - Via Firenze, 34 - I -20060 Trezzano Rosa (MI), Italia



LAST REVISION: REV. 01 DECEMBER 2017

**IMO** Via Firenze, 34 - I-20060 Trezzano Rosa (MI) - Tel. 02 909313250  
 imo@imospa.it - [www.imo-spa.com](http://www.imo-spa.com)

[www.imopronature.com](http://www.imopronature.com)

Help us to preserve the environment: dispose the packaging material as suggested on the box.